# ORBIS Tec A-Silicone

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## EN Instructions for use

Elastomer precision impression material on a polysiloxane basis, addition-curing. ORBIS Tec A-Silicone is only intended for dental purposes. A responsible and careful approach to work is assumed.

## Product description:

ORBIS Tec A-Silicone is a precision dental impression material on the basis of A-silicone that offers outstanding value for money for the highest demands. The products are manufactured in Germany from select raw materials using state-of-the-art production technology. Special fillers in the putty material make mixing easier and lead to a perfectly homogeneous result.

## Indication:

Matrix, models, fixations

## Dosage / processing:

Mixing ratio 1:1 according to volume or grams. Remove equal amounts of both components, base and catalyst, with the enclosed measuring spoon (approx. 8 ml / 15 g). Pull the material apart into a small strand, fold together again and knead well with your fingertips. Repeat a several times until the material has a homogeneous colour, though mix for at least the specified mixing time.

## Model fabrication:

Impressions can be cast after only 60 min. There are no further time restrictions. All plasters of type class 3 and 4 that meet the requirements of ISO 6873 can be used. The impressions can be cast several times.

## Work and safety instructions

The aforementioned products may only be used in accordance with the instructions for use that are provided. The user alone is liable for any other use that is not in accordance with these instructions.

- Reseal any buckets and containers tightly immediately after use.
- Avoid any contact with items of clothing because cured silicones are chemically resistant and leave stains that cannot be removed.
- In order to avoid any incompatibilities, do not combine with A-silicones from other manufacturers.
- Do not use ORBIS Tec A-Silicone together with polyether, polysulfides or condensation-curing silicones.
- Avoid any contact between the material and latex gloves since the catalyst may be damaged. We recommend vinyl gloves or PE-based gloves.
- Storage: store between 5 °C and 27 °C.
- Shelf life: 2 years from the date of manufacture, see packaging. Do not use after the expiry date.
- Processing: at room temperature between 18 °C and 25 °C

## Technical data and product properties

|             |           |
|-------------|-----------|
| Consistency | kneadable |
| Colour      | blue      |

|                                    |                |
|------------------------------------|----------------|
| Mixing time                        | 30 sec.        |
| Processing time incl. mixing time* | 1 min. 20 sec. |
| Total curing time*                 | 6 - 7 min.     |
| Shore A                            | approx. 85     |
| Aroma                              | neutral        |
| Boil-proof                         | yes            |
| Microwave-proof                    | yes            |

\*at 23 °C/73.4 °F. The times shown in the table above relate to a relative humidity of 50 +/- 10 %. It can generally be said that higher temperatures shorten and lower temperatures prolong curing.

## Warnings:

- **In case of eye contact:** avoid any contact between eyes and the impression material; if necessary, rinse eyes immediately with plenty of clear water and contact an eye specialist immediately.
- **Allergic reaction:** can trigger allergic reactions in sensitive persons. Consult a dermatologist or allergist if in doubt. If any acute allergic reactions occur during the treatment, stop the treatment with the product immediately.

## Please observe the warnings in the safety data sheet too!

Subject to errors and changes without notice.

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