MUPPY

Käppchen
starr & elastisch

D
GB
F
I
E

Patienteninformation

MUPPY zur Begradigung

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Patient information

MUPPY for straightening

MUPPY with small (biting) cap prevents tipping of the vestibular plate when the lower jaw is retracted, thereby guiding it forwards.

Apart from the negative aesthetic impression, these malformations also have functional and health consequences: Your child cannot bite off and swallow correctly, he/she also "lips" and breathes through the mouth instead of through the nose, which can lead to frequent colds, among other things. The goal of therapy is the weaning of habits and the mouth closure.

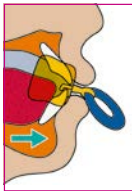
MUPPY-rules

Studies have shown that more than 90 % of children give up damaging sucking habits with the help of **MUPPY**. Your support and a few rules help your child:

- 1 **MUPPY** is a substitute for thumb and dummy. The substitution must be voluntary.
- 2 **MUPPY** must always be worn at night.
- 3 To get used to it easier children should also wear **MUPPY** a few hours during the day (when playing or watching television).
- 4 As a rule, weaning is successful within a week. However, **MUPPY** should also be worn for 6–8 weeks after successful weaning of the dummy ("recidivism").
- 5 **MUPPY** can be stored in the hygiene box.
- 6 Do not forget to regularly brush the teeth and **MUPPY**!

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MUPPY is worn similarly to a common weaning dummy. The shield lies between the lips and the small cap behind the lower front teeth. The small cap guides the lower jaw to a neutral bite position, thereby facilitating and training lip closure.

If your child is having speech therapy treatment, you should take note of the following:

- Follow your speech therapist's individual wearing and exercise instructions exactly.
- Support your child during this stage, as he/she has to feel the importance of mouth closure in order to take the exercises seriously.
- Make your child aware of the "rules of play" and supervise the necessary exercises.

Instructions for use

Before using **MUPPY** for the first time, clean it thoroughly under warm running water. Check **MUPPY** regularly for places that have been bitten through! Replace **MUPPY** when necessary.

Cleaning instructions

Clean thoroughly with a toothbrush under warm running water. Alternatively, you can also use the OROFAN® dental cleaning tablets. **MUPPY** is neither dishwasher-safe nor microwave-safe. Only clean the hygienic **MUPPY**-Box with a damp cloth.

MUPPY - designs

MUPPY small cap is available in two different sizes:

- with violet ring for deciduous dentition (size I)
- with blue ring for mixed dentition (size II)

There are also different hardness grades (elastic, rigid) available.

MUPPY small cap rigid
(transparent)

MUPPY small cap elastic
(transparent)

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