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Intended use
Material for the production of temporary crowns and bridges

Indication for use
Fixtemp is a resin based material to produce temporary crowns and bridges.

Indication
For the treatment of tooth loss and tooth substance defects.

Contraindication
Do not use in case of allergies to one of the ingredients.

Target patient group
Patients with tooth loss and tooth substance defects.

Intended users
Dentists, dental technicians

Requirements / equipment
DS 50 Injector 4:1/10:1, mixing cannula (Ø 3.2 mm)

Basic material properties
See Technical Datasheet (TDS). Technical datasheet available on request.

Processing conditions
Caution
During processing, we recommend wearing personal protective equipment, e.g. protective work clothing, suitable gloves and safety goggles. Use a suitable extraction system or face mask during finishing

Process description
Preparation
Insert the double cartridge into the injector and remove the sealing cap. Before attaching the mixing cannula, feed some material to ensure uniform discharge.

Processing
a) Direct production
In order to apply Fixtemp C&B directly from the double cartridge through the attached mixing cannula into a situation impression, which was taken before the preparation, in the area of the denture, all preparations must be blocked out and lightly insulated at undercut areas. Remove the septa in the approximal spaces from the impression. To avoid air bubbles in the applications, always keep the tip of the mixing cannula immersed in the material. Within the prescribed processing times, place the impression filled with Fixtemp

C&B in situ with the patient and hold it in position until the elastic phase is reached (approx. 2 min.). Afterwards the temporary can be removed along with the impression, but it is recommended to let it remain inside the impression for at least another 3 minutes.

b) Indirect production
To produce a temporary restoration in the dental laboratory, a thermoforming foil (e.g. Drufofen H) is pulled over the situation model from the state before the preparation in the area of the teeth to be prepared, a template is cut out and filled with Fixtemp C&B directly from the double cartridge with attached mixing cannula. To avoid air bubbles in the applications, always keep the tip of the mixing cannula immersed in the material.

30 seconds after beginning of the mixing, the thermoforming foil filled with Fixtemp C&B is put onto the insulated (e.g. using Isolat film) preparation model. After expiry of another 2 minutes, the flexible phase is reached. The thermoforming foil with the temporary restoration can be carefully removed from the model. Until complete curing, the temporary restoration must remain in the thermoforming foil for at least another 5 minutes.

Modelling
The inhibition layer formed by atmospheric oxygen can be removed with organic solvents such as isopropanol. Finish the temporary restoration after complete curing using a fine carbide bur. Coat the surface with a light-curing high-gloss varnish (e.g. Plaquit) or polish manually with a rubber polisher or goat hair brush.

Important notes
• The processing time refers to a room temperature of 23 °C ± 1 °C (73.4 °F ± 1.8 °F) and a relative air humidity of 50 %, the setting time to a body temperature of 37 °C ± 1 °C (98.6 °F ± 1.8 °F).

• Lower temperatures extend, higher temperatures shorten the specified times.

• Wear protective clothing! Set material is chemically resistant. Marks on your clothing cannot be removed.

• Never dispense into a mixing cannula that has hardened.

• After use, leave the mixing cannula on the double cartridge as a cap.

• Use only eugenol-free fixing cement.

• The temporary restoration is intended for an application period of < 30 days.

Disposal
Disposal of the content/the container according to the local/regional/national/international regulations and according to the safety datasheet.

Hazard identification / Possible dangers
Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

Cleaning
Remove the polishing dust from the finished medical device under running water using a toothbrush, for example.

Disinfection
For the disinfection of the finished temporary restoration, we recommend usual disinfectants for dental resins found in dental practices and dental laboratories.

Sterilisation
Fixtemp C&B cannot be sterilised.

Serious incidents
All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

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