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Product description
Alginate based insulation liquid for denture production
Indication
Heat-resistant, film-forming insulation agent on the basis of alginate for the field of application plaster against plaster as well as plaster against resin.
Notes
Isolat film is only suitable for indirect application.

Intended users
Dentists, dental technicians or specialist staff in the dental laboratory, orthodontists.

Requirements / equipment
Brush

Basic Material Properties
See technical datasheet (TDS). Technical datasheet available on request.

Specific Manufacturing Considerations
Caution
During processing, we recommend wearing personal protective equipment (protective work clothing, suitable gloves and safety goggles).

Process description
Further processing
Apply Isolat film onto the freshly set plaster model resp. onto a well moistened old model with a soft brush in one coating. Possible overdoses have to be removed carefully before the drying of Isolat film.

Curing
The drying time amounts to 5–10 minutes, depending on the degree of moisture. Prewarmed models accelerate the drying of Isolat film considerably. When reusing older dry models it is recommended to boil out the model and to coat it newly with Isolat film.

Important notes
- Dilution is possible with water (better distilled water). Close well after use.

Cleaning
Before first use on the patient, the dental applications should be cleaned to remove any residual insulating material that may have adhered.

Lot number / Expiry date
The batch number and the use-by date are located on both the outer packaging and the product. When submitting a complaint about a product, please always quote the batch number. Do not use the product after expiration of the use-by date.

Hazards identification
Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

Disposal
Disposal of contents/container in accordance with local/regional/national/international regulations and according to the safety data sheet.

Serious incidents
All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

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